Centrífuga para colados
CEN MOTOR SM-2

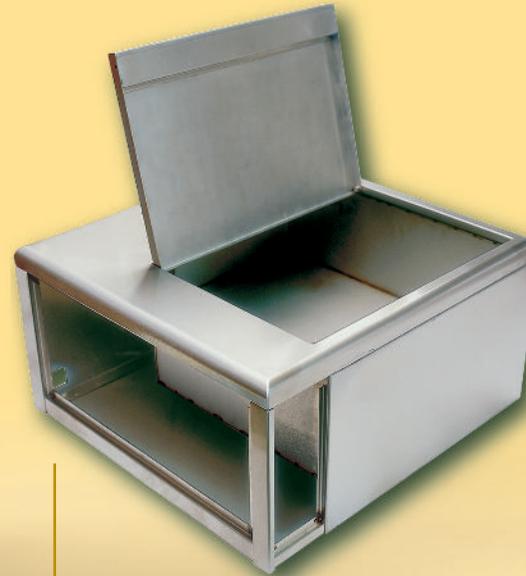
CEN MOTOR SM-2 (ACERO INOXIDABLE)
SM-2 CEN MOTOR (STAINLESS STEEL)
 CEN-MOTOR SM2 (ACIER INOXYDABLE)

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With our SM-2 model we are pleased to launch a new generation of casting machines. On its design we have combined MESTRA broad experience in this kind of machines and the

latest technological advances. As a result we offer a modern practical casting machine, without forgetting the usual robust, reliability and quality features so distinctive of our products.

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The bodywork is completely made in stainless steel. Casting cabin fixation to the main body is made by an elastic joint that reduces operating vibrations. Machine external dimensions are reduced so it can be easily placed in the lab.

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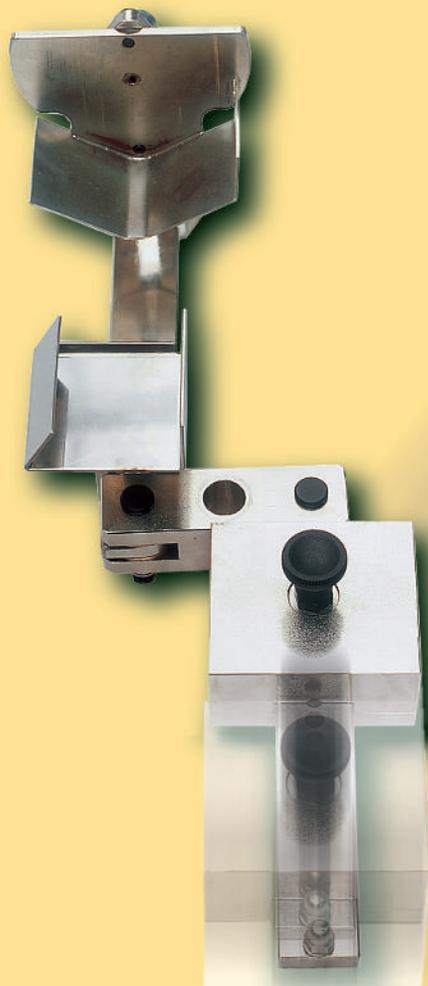
Mobile parts have been reduced to the minimum thus improving mechanic reliability and sonority. The main shaft -made in high resistant steel- is guided by two ball bearings that produce a soft and shaft operation.

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The counterweight slides softly over a graduated guide with all different casting ring dimensions that can be mounted. For blocking and unblocking manoeuvres a knob is used.

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SM-2 has a robust stainless steel cabinet available (not included) that transforms the over table model into a full-unit machine. Furniture shelves are very practical in order to store tools used in casting operations.

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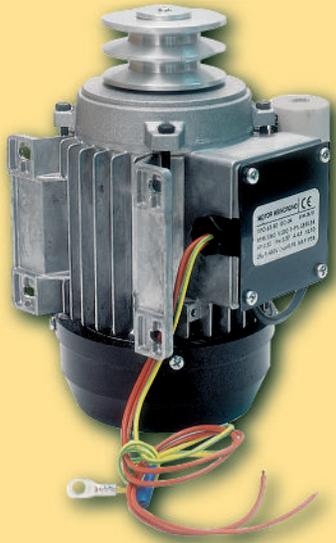
1/2 HP motor has been specifically designed for this application. It confers a high starting torque as well as reduced dimensions. Bodywork anchorage is made by rubber silentblocks.

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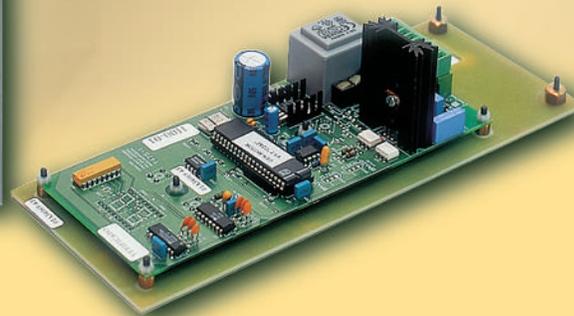
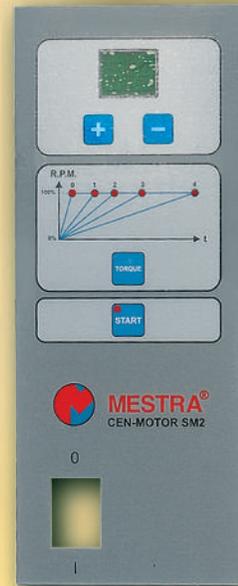
The electronic control regulates motor torque by microprocessor. Power is dosed according to the starting up chosen up to five options available. Operation time can also be easily selected among 10 and 60 seconds, watching it on a display.

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
When the machine is working, cabin door is blocked by a safety closing that avoids possible accidents.

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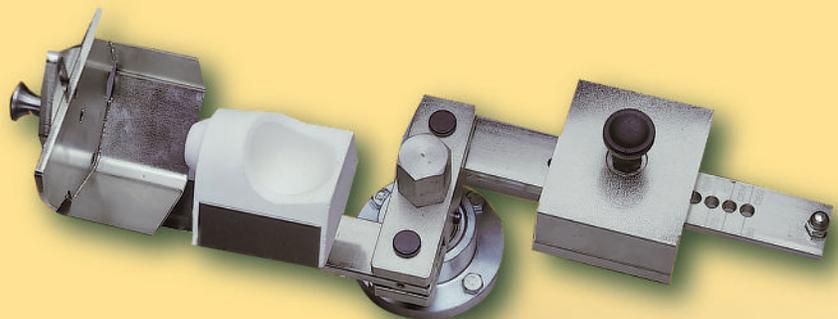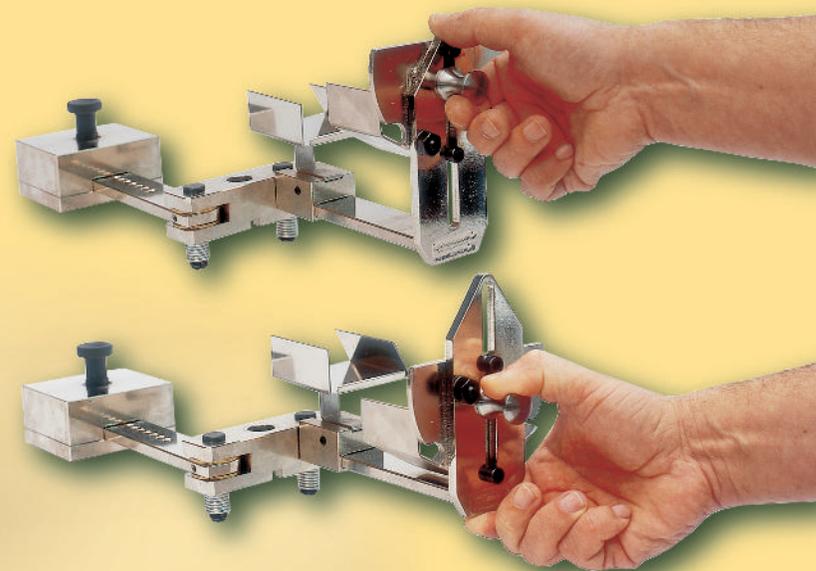
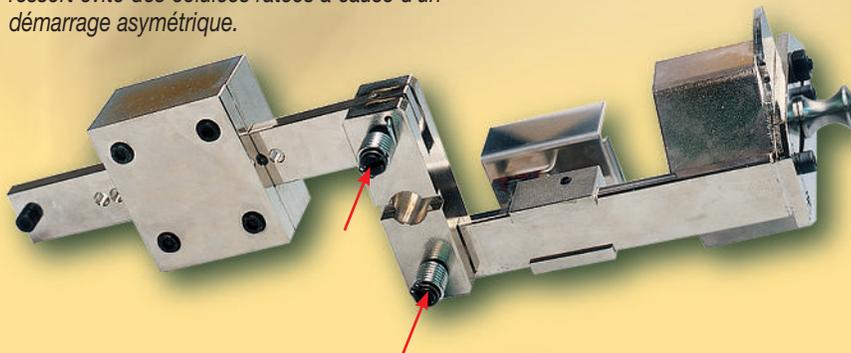
Light arm that reduces starting inertia and operating vibrations. Completely made of high resistant steel with nickel coat.

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Spring automatic withdrawal system to avoid coating losses when asymmetric starting.

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Casting ring support allows mounting up to 100 mm casting rings. By a simple height regulating system an optimum line-up between crucible muzzle and dish is obtained.

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