smartbox X2

Betriebsanleitung
Operating instructions
Mode de fonctionnement
Istruzioni sul funzionamento
Instrucciones de servicio

AMANNGIRRBACH

Deutsch	04 - 10
English	11 - 17
Français	18 - 24
Italiano	25 - 31
Español	32 - 39

SMARTBOX X2

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SMARTBOX X2

Translation of the original operating instructions

- ❶ LC-Display
- ❷ Rotary knob
- ❸ Plaster container
- ❹ Plaster container cover
- ❺ Water tank
- ❻ Water tank cover
- ❼ Balance
- ❽ Plaster outlet connecting piece
- ❾ Connecting piece's screw
- ❿ Screwcap
- ⓫ Angle piece

PRODUCT DESCRIPTION

Smartbox X2 uses storable programs for dispensing water and plaster. All types of dental plasters can be used.

Smartbox provides consistently high quality dispensing results and is easy to use. The special dispensing technology allows plaster to be finely sprinkled into the mixing beaker, thus improving mixed materials' homogeneity noticeably. Smartbox distinguishes itself through its robust construction. According to the experts, the water should be dispensed before the plaster, via the water tank integrated within the upright part of the unit. The 20 programs provide the opportunity to select and directly start almost any plaster / water dosage used in the laboratory at the touch of a button Smartbox X2 was designed to be as ergonomic and user-friendly as possible.

This is apparent in the Smartbox X2 controls which only consist of the multifunctional rotary knob and the two-line display.

Each dispensing program comprises the following settings:

Amount of plaster:

- _ Amount of plaster may be defined in 5 gram steps, from 50 g to a maximum of 1000 g.

Plaster/water ratio:

- _ Plaster / water ratio is defined in 1ml steps per 100 g of plaster and automatically converted to the required amount of plaster.

Program identification:

- _ Each program can be individually designed with its own name.
- _ This program identification may consist of no more than 12 letters.
- _ All data are entered via the multifunctional rotary knob.

Soaking time:

- _ Time for soaking of plaster in water - up to a maximum of 30 s.
- _ A short acoustic signal indicates when the time is up.

INSTALLATION

After unpacking, immediately check the machine for possible damage incurred during transport. Should the machine have been damaged during transport, please let the supplier know immediately.

Machine is delivered ready for operation.

Filling the water tank:

After unpacking, rinse the water tank and fill with water, then hang it up on top of the stand on the unit. Ensure that water tank snaps into place correctly on the support **(A)** - **(D)**. The flexible water suction hose protruding from the stand is then connected to the circular opening on top of the water tank. Then close the opening again with the black cap.

Filling the plaster container:

CAUTION: Before filling the plaster container for the first time, make sure that the cap is on the plaster outlet. The cap prevents the plaster from flowing out.

The stainless steel plaster container **(E)** is designed for a filling quantity of approximately 8 kg. Plaster container includes a powder level monitoring system **(F)**. Should the plaster level drop to the minimum level of approximately 500g, this will be shown in the display.

If the container is re-filled before having been completely emptied, then it is not necessary to seal the plaster outlet.

The cap contains a magnet and may be fastened to the back plate of the machine when not in use. In case of prolonged downtime, we recommend replacing the cap back onto the plaster outlet once more, in order to prevent moisture from penetrating into the plaster container!

HANDLING

To start the Smartbox X2, first switch it on by means of ON/OFF switch on the back of the machine. On its front, Smartbox X2 is equipped with a two-line LC display ❶ and the multifunc-

tional rotary knob ❷ which enables all Smartbox X2 functions.

After switching-on, the machine is immediately ready for operation.

On initial activation, the Smartbox X2 language selection will be displayed. Select the desired language and acknowledge by pressing the button. When the machine is switched on at any other time after this, the display will show the dosage program which was used last.

LANGUAGE CONFIGURATION

Menu navigation of the Smartbox X2 is multilingual (German, English, French, Italian and Spanish). In case the language displayed during installation is not your preferred language, you can change it within program P24.

- _ Select program P24 by turning the rotary knob.
- _ Keep the rotary knob pressed down for 1.5 s.
- _ Menu switches to language selection.
- _ Select desired language by turning the rotary knob.
- _ Press "Enter" to confirm your choice.

Machine is now ready for operation.

STORAGE OF DISPENSING PROGRAMS

Smartbox X2 is programmable on two levels: The working level with dispensing programs P01 to P20 and the programming level for determining the dispensing parameters for the corresponding dispensing program.

Switching between the two levels takes place

by pressing rotary knob for 1.5 s and keeping it pressed. Change the respective configuration by pressing and turning the rotary knob.

SWITCHING BETWEEN THE TWO PROGRAM LEVELS

1. Switching from the working level to the programming level:

Keep pressing rotary knob for 1.5 s to get to the programming level where you can change the mixing parameter.

2. How to switch from the programming level to the working level:

Press the rotary knob and keep it pressed for 1.5 s in order to change from the programming level to the working level at any time and at any position. Any changes of dispensing parameters made at the programming level are automatically saved when you leave the programming level.

MENU STRUCTURE

P01	Alpenrock		
250g		100:20	
Amount plaster			
		250g	
Amount of water			
	20ml:100g		
Soaking time			
		30s	
Program name			
	Alpenrock		

PROGRAMMING

1. Program selection:

By turning the rotary knob, select the program you want to change.

2. Switching to programming level:

Press rotary knob for 1.5 s to switch to programming level.

3. Setting the amount of plaster to be dispensed:

The amount of plaster to be dispensed can be set to the desired value (from 50 to 1000g) in 5g steps, by turning the rotary knob.

Press the rotary knob briefly to confirm your input.

Program now switches to the next dispensing parameter.

4. Setting of water/plaster ratio:

Next parameter to be set is the water ratio per 100g of plaster. By turning the rotary knob, the desired water ratio per 100 g of plaster is selected and confirmed by briefly pressing the knob.

5. Starting the soaking function:

Next, the soaking time function may be switched off or switched on, with a time indication of 5 to 30 s. Confirm value by briefly pressing the knob.

6. Program identification:

The program identification can be put into the last field. Continue turning the rotary knob letter by letter until the correct letter has been found. Confirm by briefly pressing the knob, cursor then automatically skips to the next space. Identification with up to 12 letters may be used. If not all 12 spaces are needed, fill the left over spaces with blanks by pressing the rotary knob several times quickly or abort the identification mode by pressing the rotary knob for 1.5 s.

For your information: Settings always refer to one program. Up to 20 individual programs may be stored.

DISPENSING

Make sure that the cap is not covering the plaster outlet.

1. Select mixing beaker:

Select the correct size of mixing beaker for the required amount of water and plaster and make sure that mixing beaker is placed centrally under plaster outlet.

2. Select dispensing program:

Select desired dispensing program (P01 to P20) by turning the multifunctional rotary knob **(G)**.

3. Dispensing step:

Start dispensing by pressing the rotary knob **(H)**.
Water will be dispensed before the plaster
(recommended by experts) **(I) (J)**.

LC display indicates both dispensed amount of water and plaster in ml or g.

An acoustic signal indicates the end of dispensing step.

If the dispensing parameters include soaking time, an acoustic signal will ring out to signal the end of the set time.

MANUAL DISPENSING OF WATER AND PLASTER

To dispense plaster and water manually, use additional programs P21 "Dispensing of plaster" and P22 "Dispensing of water".

Procedure is described below for program P21
"Dispensing of plaster". Procedure is the same

for P22 "Dispensing of water":

Start dispensing of plaster by pressing the rotary knob. First, the scales are tared. During dispensing of plaster keep the rotary knob pressed down until the desired amount has been dispensed. This way the desired amount of plaster can be dispensed without any need for programming.

When rotary knob is released, plaster dispensing stops. If within 5s, the rotary knob is pressed down again, dispensing can be continued. Otherwise, weighing function in this program extinguishes automatically after 5s.

By turning the rotary knob the weighing function is stopped prematurely, enabling the user to switch to another program.

For manual dispensing of water, proceed the same way in program P22 "Dispensing of water".

P21		D	i	s	p	e	n	s	i	n	g								
				P	l	a	s	t	e	r									
Scales:																			
																	TARE		
Scales:																			
																	0.	0g	
P22		D	i	s	p	e	n	s	i	n	g								
				W	a	t	e	r											
Scales:																			
																	TARE		
Scales:																			
																	0.	0g	
P23		S	c	a	l	e	s												
Scales:																			
																	TARE		
Scales:																			
																	0.	0g	

OPERATING THE WEIGHING FUNCTION

Within the program P23 "Weighing", any item of up to 2.5kg may be weighed.

Press rotary knob to first tare the weighing scales and activate weighing function.

Put the item to be weighed on the platform - weight will be indicated in LCD.

If now an item is on the weighing machine, weighing machine may be tared again by pressing rotary knob again (reset the tare at zero) so that another item may be weighed together with the first one.

If weight doesn't change within 10 s, weighing function stops automatically and switches to program selection.

SERVICING

Smartbox X2 is designed to be maintenance-free. Simply ensure the machine is kept clean.

Cleaning of plaster outlet:

Plaster outlet connecting piece ⑧ can be removed from the machine. To do this, unscrew the connecting piece's screw ⑨, take off the outlet unit ⑧ and clean. Take care not to damage the steps at the inner side of the connecting piece during cleaning operation.

Cleaning of water outlet:

Water outlet can also be cleaned. To do so, unscrew screwcap ⑩ and cautiously pull forward the angle piece ⑪. In this position, any residues may be removed.

For more thorough cleaning or replacement, the angle piece can be withdrawn from the hose.

After cleaning replace the angle piece back onto the hose and secure with screwcap. In doing so, check the direction of the water outlet.

Please ensure that no fluid enters plaster container.

Prior to transporting the Smartbox X2, ensure the plaster container and water tank are completely empty. Always make sure that Smartbox X2 is only transported within its original packaging. In case the original packaging does not fit any more or has been discarded, please refer to local retailers.

CLEANING ADVICE:

Prior to cleaning, disconnect the machine. For cleaning surfaces use only soft, slightly damp cloths.

Avoid scouring, especially on polished surfaces. Caution when using stainless steel cleaning equipment for the silo!

These detergents may corrode rubber and polished surfaces and leave stains!

Never clean Smartbox X2 with compressed air or steam cleaner!

INSTRUCTIONS AND FAILURE REPORTS

"Please refill plaster"

If the plaster level falls below the minimum level of approximately 500g, a plaster re-fill will be requested.

If however, a small amount of plaster is required, dispensing process may be started by a second acknowledgment of filling level sensor unit.

CAUTION - Residual amount in the container may be too low!

P	l	a	s	t	e	r		l	e	v	e	l	?		
S	t	a	r	t	?								y	e	s

If the plaster tank lid is not closed properly, the status signal "Plaster level?" will appear even when there is 500g of plaster inside. The fill level sensor will react early. Always ensure the plaster tank lid is properly closed.

"Dispensing process stopped"

If after starting or during a dispensing operation weighing plate does not detect any change of weight after 4s, dispensing operation will stop.

Possible reasons:

- _ One of the containers, plaster or water, is empty.
- _ Plaster or water outlet clogged
- _ Cap is still on plaster outlet
- _ Hose in water tank is clogged or bended
- _ Remove any possible residual material under weighing plate → the site under the weighing plate.

CAUTION - do not pull the weighing plate upwards!

If however, it is still not possible to start or end the dispensing process, please contact your relevant service department.

Accessories and extent of delivery:

Order no.:

116170	Smartbox X2, power cable, water tank, stand cover, weighing platform cover, plastic shovel
--------	--

Spare parts:

Order no.:

116175	Water tank
116176	Water tank cover
116178	Stand cover
116180	Plaster outlet cap
116181	Angle piece water outlet
116183	Plaster container cover

WARNING AND SECURITY ADVICE:

Smartbox X2 has been developed for use in dental laboratories. Install and operate it according to user manual.

The manufacturer is not liable for damage caused by unintended use or improper handling.

In addition, the user is directly responsible for checking the machine prior to use for suitability and usability for the designed purposes, unless these are listed in user manual.

- _ **Do not clean the Smartbox X2 with solvent-containing fluids!**
- _ **Always keep plaster container and plaster outlet-connecting piece clean and dry!**
- _ **Keep the weighing machine clean and dry!**
- _ **Only fill the plaster container with dental plaster - never with implant mass or other materials!**
- _ **Never clean the Smartbox X2 using compressed air!**
- _ **Prior to any transport, ensure the plaster silo and water container are empty and clean.**
- _ **Never expose Smartbox X2 to high humidity (e.g. steam cleaner, ultrasonic cleaning device or washbasin)!**
- _ **Never operate Smartbox X2 water pump without water.**
- _ **Ensure that the water tank is filled with pure water only, without chemical additives.**

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Amann Girrnbach AG
Amann Girrba AG
Amann Girrba GmbH